

FRFW-556653

Description

เป็นเฟืออกอ่อนสำหรับใช้ตามกระดูกสะโพกสำหรับผู้ที่มีการบาดเจ็บที่บริเวณสะโพกใช้ยึดตรึงกระดูกสะโพกขณะเคลื่อนย้าย

SAM MEDICAL PRODUCTS®



SAM Pelvic Sling™ II



Specification

- ◆ เฟืออกที่ผลิตจากวัสดุอย่างดีในการออกแบบ และพัฒนาให้ใช้งานได้ง่ายเพียงสามขั้นตอน มีความกะทัดรัด ทนความชื้นและอุณหภูมิที่สูง
- ◆ วัสดุมีความยืดหยุ่นสูง ไม่ยืดหรือเสียรูปทรง สามารถทำความสะอาดได้และนำกลับมาใช้ใหม่ได้ ป้องกันปัญหา การตกร้างของคราบสกปรก

◆ มีขนาดมาตรฐานสามารถใช้ได้กับสะโพก ถึง 3 ขนาด

Small - 69cm to 119cm

Standard - 81cm to 127cm

Large - 91cm to 152cm

◆ สามารถเอ็กซ์เรย์ได้ทันทีโดยไม่ต้องถอดเฟืออกออกจากตัวผู้บาดเจ็บ


◆ ได้รับการทดสอบและพิสูจน์ทางวิทยาศาสตร์และทางการแพทย์ว่าปลอดภัย

และมีประสิทธิภาพที่ทำให้กระดูกสะโพกไม่เคลื่อนที่


Applies in 3 Easy Steps no trimming, no cutting, no guessing



1. Remove objects from patient's pocket or pelvic area. Place SAM Pelvic Sling II black side up beneath patient at level of trochanters (hips).

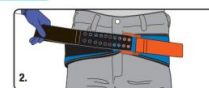

2. Place BLACK STRAP through buckle and pull completely through.

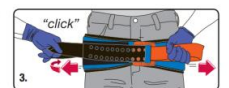

3. Hold ORANGE STRAP and pull BLACK STRAP in opposite direction until you hear and feel the buckle click. Maintain tension and immediately press BLACK STRAP onto surface of SAM Pelvic Sling II to secure. You may hear a second click as the sling secures.